

Perifix® Accessories

Perifix Catheter Connection System

Epidural Filter


- 0.2µm particulate filter
- Low profile design for patient comfort

Catheter Connector

- Securely and easily fixes catheter in place
- Audible "click" verifies solid connection has been made
- One-handed, smooth operation

PinPad

- Helps reduce disconnects and access errors
- Lets patients move freely while keeping catheter connector and filter securely in place

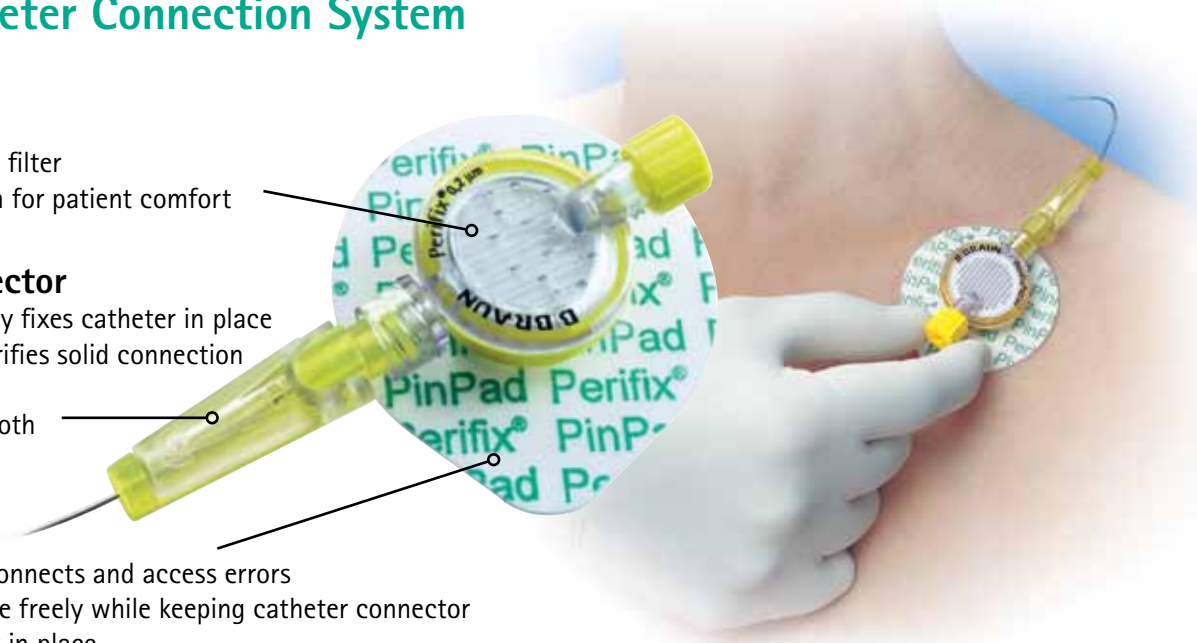

Perifix LOR Syringes

- Consistent, reproducible feel time after time
- Smooth, low-friction plunger action
- Available in 8mL plastic syringe and 5mL or 10mL glass syringes, luer slip or luer lock



On-going Professional Support

Delivering state-of-the art healthcare demands more than technological innovation. It requires on-going educational initiatives and clinical support that can help you and your

staff maintain critical skills and master innovative techniques. That's why healthcare providers around the world turn to B. Braun for comprehensive professional support.



Clinical and Technical Support

Our technical support staff is also available to answer your calls about our regional anesthesia products. To speak to a trained clinician about product information or technical issues, call **800-854-6851**.



Resource Library

B. Braun offers a wide array of educational resources to help you learn more about regional anesthesia techniques. These resources have proved to be valuable teaching tools for your staff and patients and are available from your B. Braun representative or at our website, www.bbraunusa.com.



Online Tutorials

Log on to www.nerveblocks.net and learn at your own convenience! This multimedia online tutorial allows you to update your knowledge of peripheral regional anesthesia principles and procedures. Available 24/7, the tutorial also features a discussion forum and event schedule.

Epidural Anesthesia Products

Help improve procedural outcomes and reduce risks



High quality, reliable epidural anesthesia devices from the leader in Regional Anesthesia.

B. Supported.

Sharing Expertise®.
It's more than a philosophy.

It's our promise to the healthcare community and ourselves to continuously seek knowledge. To expand it. To apply it. To share it. All to help improve and save lives.

This is Sharing Expertise.
This is B. Braun.

Call **1-800-227-2862** for your B. Braun representative and for more information on our Peripheral Nerve Block, Spinal and Custom products and programs.

B. Secure.



Mixed Sources
Product group from well-managed
forests, controlled sources and
recycled wood or fiber
www.fsc.org Cert no. COC-1000-000000
© 1996 Forest Stewardship Council

B. BRAUN
SHARING EXPERTISE

www.bbraunusa.com

B. BRAUN
SHARING EXPERTISE

Choose B. Braun. Your Reliable Partner in Epidural Anesthesia

As the worldwide leader in regional anesthesia products, B. Braun has become recognized for superior performance and quality that you can trust. The results — products engineered to provide consistent feel and superior outcomes, delivered the way you want them.

Perifix® epidural products combine precision, quality, and outstanding control — features you need to meet today's exacting pain control demands.

Whether you are relying on our needles, catheters or syringes, the advanced design and unsurpassed quality of the Perifix family of products helps ensure safe and reliable epidural procedures.

And because B. Braun is a world leader in epidural products, you can turn with confidence to Perifix.



Trays are available in standard or custom configurations. Contact your B. Braun sales representative to design your own.



Perifix® Epidural Needles

Quality and safety

Identifying the epidural space requires skill, experience and a consistent feel. Perifix needles feature the qualities necessary to meet these needs.

- State-of-the-art design
- Meticulous construction
- Wide selection of styles and sizes, including short length needles

The Perifix 17G epidural needle was the most durable of the needles tested¹ in a stiffness and malleability test²

Needle	Buckling Force (Newtons)
Perifix 17G, O.D. 0.0589 in.	57.49
BD 17G, O.D. 0.0577 in.	40.62
Perifix 18G, O.D. 0.0532 in.	40.32
AMI 18G, O.D. 0.0502 in.	27.07
Manan 18G, O.D. 0.0499 in.	28.58




Perifix Safety Epidural Needle

The Perifix Safety Epidural Needle uses B. Braun's proven safety shield which is designed to reduce needlestick risks and increase compliance without requiring a change in technique.

According to the study from Greene, ES, et al., 7.91 contaminated percutaneous injuries (CPI) occurred per 100,000 units.³ These injuries could have been avoided by using safety devices.

Advance the yellow plastic housing to the tip of the needle to engage the safety shield

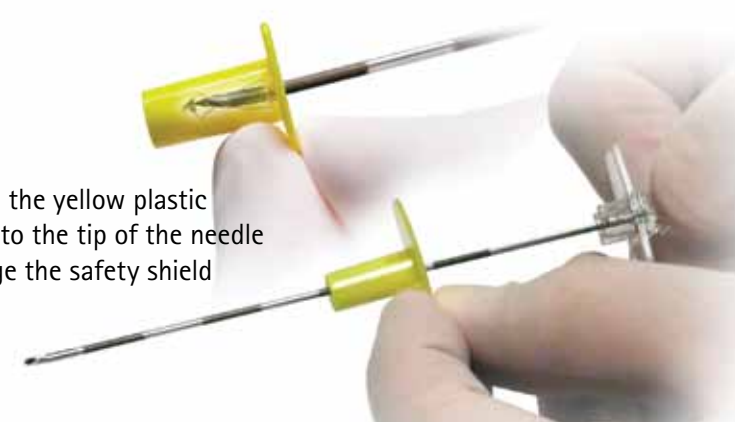

Perifix® Epidural Catheters


Reliable Performance and Improved Patient Outcomes

Superior product performance has been the hallmark of B. Braun epidural catheters for more than 25 years.

Because of this, anesthesia providers have come to trust Perifix epidural catheters for reliable performance and improved patient outcomes.

Perifix catheters are:

- Kink and stretch resistant
- Easy to thread and remove
- Latex-free
- Easy to anchor with the Perifix Catheter Connector

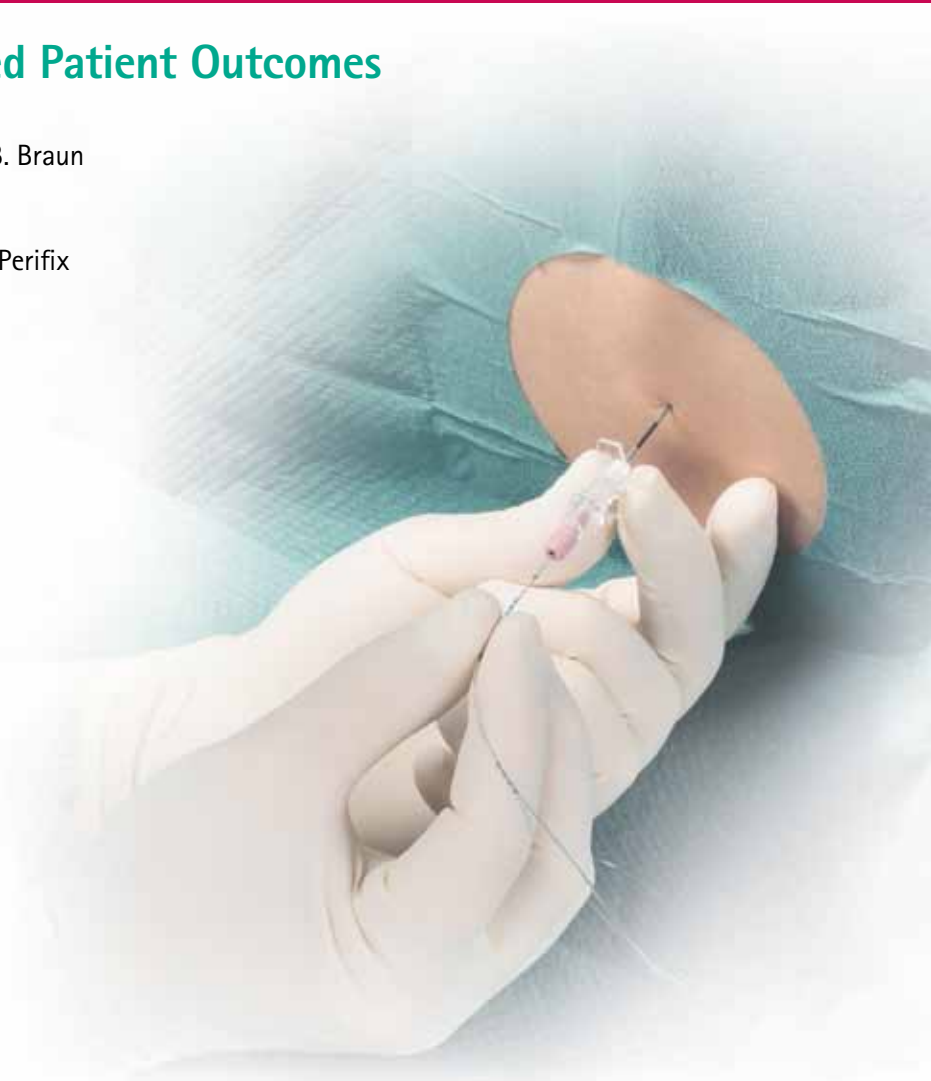

Perifix FX Springwound
Available in 19G
open or closed-tip

Perifix Standard
Available in 19G
and 20G open
or closed-tip

Perifix SoftTip
Available in 20G
closed-tip that
can be used
with 18G and
17G needles

Perifix ONE
Available in 20G
and 24G closed-tip.
Features six lateral
side ports



For Pediatric use only

1. Dunn, SM, Goolishian, WT. All epidural needles are not created equal. *Anesth Analg.* 1996; 83(3):659.
2. Dunn, S., et al. A fractured epidural needle: case report and study. *Anesth Analg.* 1992;75:1050-1052.
3. Greene, ES, et al. "Multicenter study of contaminated percutaneous injuries in anesthesia personnel. *Anesthesiology* 1998; 89: 1362-1372